

reveal[®]
CLEAR ALIGNERS

 **HENRY SCHEIN**[®]
DENTAL WAREHOUSE

Rely on Us[™]




How To Guide
3Shape Integration

TOLLFREE 0800 111 796

www.henryschein.co.za

reveal@henryschein.co.za

Step 1 Login to 3Shape Communicate, click on the “Connections” tab and search for your Aligner Lab via the lab’s main contact email or name.

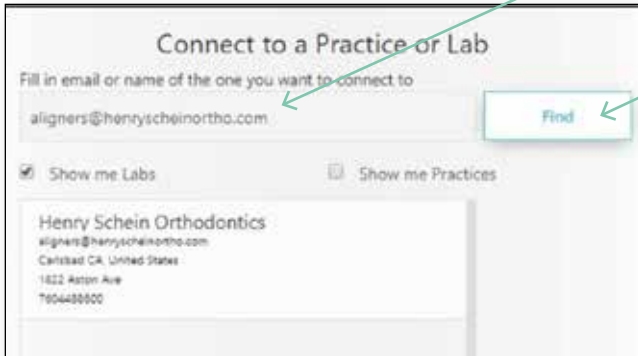


Step 2 Under Settings, choose “Lab Connections” and select “Update”.

The Henry Schein Ortho lab account will appear as a lab on the practice’s TRIOS.

Step 3 Logout.

Step 4 Login to DDX, click on the drop-down arrow on the top right hand side of the page, to choose Store.

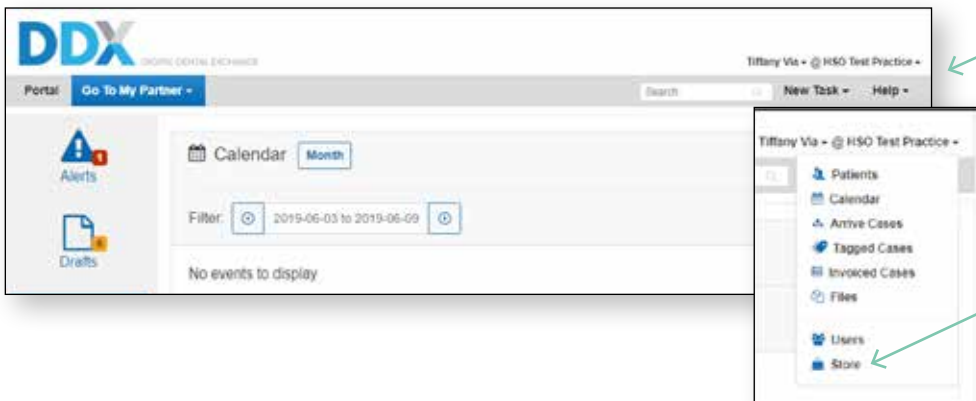

Step 5 Scroll down and find TRIOS 3Shape Communicate and click on “Enable”.



Step 6 Click on “Configure your add-on”



Step 7 Click on “Authorise” and type in the email and password of your Communicate account.



Step 8 Now the two networks are connected! Before performing a TRIOS scan, select Reveal Clear Aligners as the receiver and DDX will create a Draft case.

reveal[®]
CLEAR ALIGNERS

 **HENRY SCHEIN**[®]
DENTAL WAREHOUSE

TOLLFREE 0800 111 796

reveal@henryschein.co.za | www.henryschein.co.za